



Triceram® Liner

For top aesthetic results

Optimal Bonding Strength

A new type of colour intensity

Gives the illusion of depth, three dimensionally



With the new Triceram® Liners from Esprident it is now possible to mask the white colour of the zirconium oxide in the desired tooth shade within the first working step. The individual tooth colour creates an illusion of depth and makes the restoration appear vital and natural. The new colour intensity of the liner provides a base for the subsequent aesthetical build-up.

Special Qualities

- It is possible to mask the white colour of the zirconium oxide with the desired tooth shade within the first working step and create top aesthetical results.
- The Triceram® Liner makes the tooth shade more radiant, therefore intensifying the natural colour effect.
- It is possible to reproduce all the shades of the classical VITA* shade guide quickly and easily using just six basic colours.
- The adapted thermal characteristics of Triceram® Liner guarantee the compatibility between the zirconium oxide framework and the Triceram® ceramic.
- Optimal bonding strength between zirconium oxide framework and the Triceram® ceramic.

* VITA is a trademark of the company VITA Zahnfabrik, Bad Säckingen

Quick and Easy

Individual tooth shades

Due to combination of colours

It is possible to reproduce all the shades of the VITA* shade guide by mixing just six basic Triceram® Liner colours. Simply mix the Triceram® Liner according to the desired tooth shade (see mixing example) together with the B.O.L.- liquid.

Mixing recommendation

Tooth shade	L 1	L 2	L 3	L 4	L 5	L 6
A1	1/3				2/3	
A2	2/3				1/3	
A3	1					
A3,5	3/4					1/4
A4	1/2					1/2
B1		1/3			2/3	
B2		2/3			1/3	
B3		3/4			1/4	
B4		1				
C1			1/3		2/3	
C2			2/3		1/3	
C3			3/4		1/4	
C4			1			
D2				2/3	1/3	
D3				3/4	1/4	
D4		1/3		2/3		

Program of delivery

■ L 1	20 g	Order-No. 292-513-40
■ L 2	20 g	Order-No. 292-523-40
■ L 3	20 g	Order-No. 292-533-40
■ L 4	20 g	Order-No. 292-543-40
■ L 5	20 g	Order-No. 292-550-40
■ L 6	20 g	Order-No. 292-567-40
■ Universal B.O.L.-liquid	20 ml	Order-No. 299-180-40
■ Universal B.O.L.-liquid	100 ml	Order-No. 299-180-80

Our technical advisors will be pleased to help you with any question regarding the use of our products.

Hotline **+49 72 31 / 803-440**

Fax **+49 72 31 / 803-321**

Orders **+49 72 31 / 803-210**

Informations and instructions for using Esprident ceramics systems can be found on the internet at

www.esprident.com

Date of Information: 07/04

-we reserve the right to make changes-

Picture references:

Dirk Ahlmann (DCS Schulungszentrum) · 65779 Kelkheim · Germany

ESPRI DENT

Esprident GmbH · aesthetic dental products · Turnstraße 31 · 75228 Ispringen · Germany
A Dentaureum Company · Telefon +49 72 31 / 803-0 · Fax +49 72 31 / 803-321
www.esprident.com · E-Mail: info@esprident.com